


TRU-CLOSE® Thoracic Vent


For treatment of pneumothorax

- For treatment of spontaneous, traumatic, or iatrogenic simple pneumothorax
- The self-contained vent allows complete ambulation during resolution
- The syringe-activated drain port allows aspiration of small amounts of fluid

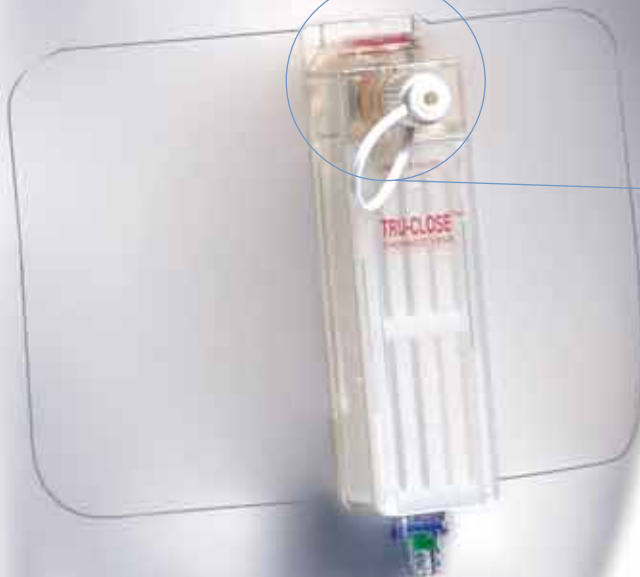

The red signal diaphragm deflects upwards when the catheter tip enters the pleural space. It then continues to fluctuate with respiration as the intrapleural pressure varies from $> +2\text{mm Hg}$ to $< -2\text{mm Hg}$.

When the signal diaphragm stays in the down position and the patient is asymptomatic the pneumothorax may be resolved.

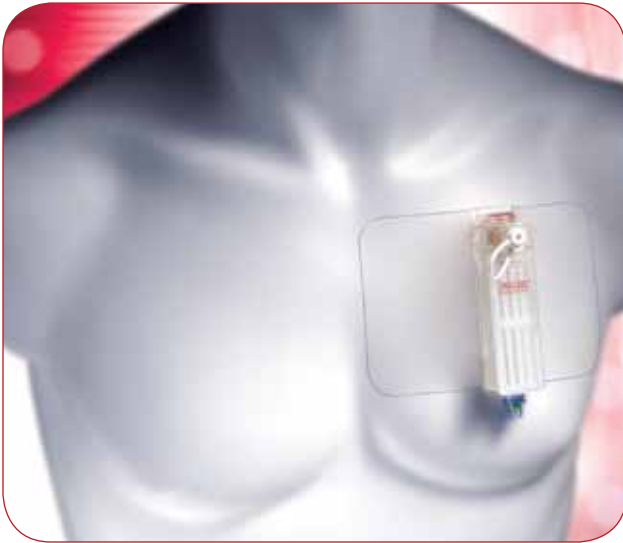

- For treatment of spontaneous, traumatic, or iatrogenic simple pneumothorax
- The self-contained vent allows complete ambulation during resolution
- All the components necessary for immediate insertion and evacuation are contained in the procedure tray
- The syringe-activated drain port allows aspiration of small amounts of fluid



Procedure Tray contains:

- Thoracic vent
- Trocar
- Aspiration cannula
- Suction tubing set
- 60cc syringe
- 3cc syringe
- Safety needle, 25G x 5/8"
- Safety needle, 22G x 1 1/2"
- Scalpel
- Halstead Forceps
- Gauze sponges (2)
- Fenestrated drape
- CSR wrap
- Occlusion Plug

TRU-CLOSE Thoracic Vent Procedure Tray

Product Codes	Cath. O.D. (Fr.)	Cath. Length (cm)
TV11-10	11	10
TV11-13	11	13
TV13-10	13	10
TV13-13	13	13

Over-the-Wire Insertion Cannula for TV11-13

Code	Units/box
OTW11-13	5

Suction Tubing Set

Code	Units/box
TVST	5

Occlusion Plug

Code	Units/box
TVOCP	5

Your Local Representative

Ordering Information: Call 1-800-538-7374

For more information about UreSil products, please contact our sales department.

Made in the U.S.A. by UreSil, LLC.

5418 W. Touhy Ave., Skokie, IL 60077

Tel: (800) 538-7374 Tel: (847) 982-0200 Fax: (847) 982-0106

UreSil, Ureflex and TRU-CLOSE are registered trademarks of UreSil, LLC.

U.S. patent no. 6,770,063

SF 08-11 TCTV

EU: PDS; Surrey GU2 7YG, Tel: 00 44 1483 685420

